# EXPRESSfit<sup>®</sup>

Fitting System  
Installation Guide

 **SONIC**  
innovations

English

Deutsch

Français

Español

Dansk

Nederlands

Norsk

Türkçe

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Português

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# Table of Contents

## ENGLISH

EXPRESSfit® Fitting System Installation Guide..... 4

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# EXPRESSfit® Fitting System Installation Guide

This guide contains the installation instructions for the EXPRESSfit® Fitting System. To learn more about using the EXPRESSfit® Fitting System software, refer to the Online Help. Online Help can be accessed via the Help menu, or by clicking the “?” icon located at the top of each screen in EXPRESSfit®.

## System Requirements

For successful EXPRESSfit® Fitting System installation and operation please make sure your computer meets the following system requirements:

<b>Processor</b>	Pentium 3-Compatible, 750 MHz or better
<b>Operating Systems</b> Visit: <a href="http://windowsupdate.microsoft.com">http://windowsupdate.microsoft.com</a> to install the latest operating system and software updates	Windows XP Professional (Service Pack 3 or later); Windows Vista (32 bit only) Home Basic, Home Premium, Business; Windows 7 (both 32 and 64 bit) Home Premium, Professional, Ultimate
<b>System RAM</b>	1 GB Required, 2+ GB recommended Additional RAM recommended for large fitting system databases
<b>Free Hard Disk Space</b>	1 GB Additional disk space required for large fitting system databases
<b>Screen Resolution</b>	1024x768 minimum
<b>CD ROM Drive</b>	Required
<b>Programming Interfaces</b>	HI-PRO ( <b>not recommended</b> for Velocity™, Endura™, ion™, Touch®, or Pearl™ products) NOAHlink™ EXPRESSlink™
<b>Serial COM Port</b>	Required for HI-PRO Can be replaced by a USB-to-Serial converter
<b>USB Port</b>	Required for NOAHlink™ or EXPRESSlink™ USB Port on the computer or powered USB hub
<b>NOAHlink™ Driver</b>	NOAHlink™ Properties: 1.50.04 or later NOAHlink™ Kernel: 1.50.03 or later
<b>Internet Connection</b>	Recommended Required for EXPRESSfit® Standalone installations
<b>Internet Explorer</b>	Version 7.0 or later
<b>NOAH Software</b>	Required for EXPRESSfit® for NOAH Fitting System Version 3.6.1 G required, version 3.7 recommended

## Hardware Installation Instructions

### If using NOAHlink™

For new NOAHlink™ installations, the NOAHlink™ software must be installed before installing the EXPRESSfit® Fitting System software. After installing the NOAHlink™ software, leave the NOAHlink™ unit attached to the computer, then install the EXPRESSfit® Fitting System.

### If using EXPRESSlink™

For new EXPRESSlink™ installations, it is strongly recommended that the EXPRESSfit® Fitting System software is installed prior to attaching a new EXPRESSlink™ to your computer. The drivers are automatically installed as part of the EXPRESSfit® Fitting System installation.

If an EXPRESSlink™ has previously been used with the computer, it is important to follow the instructions during the EXPRESSfit® Fitting System installation to update the drivers by unplugging the EXPRESSlink™ when prompted.

Attention Windows XP Users: You may encounter a message stating that the EXPRESSlink™ software has not passed Windows Logo testing. If you get this message, click the “Continue Anyway” button to proceed with the installation. If you have successfully connected EXPRESSlink™, the power indicator on the front panel turns bright green.

# Software Installation Instructions

## EXPRESSfit<sup>®</sup> for NOAH Fitting System

### Steps to Install the EXPRESSfit<sup>®</sup> for NOAH Fitting System:

1. Make sure NOAH is not running on the computer.
2. Insert the EXPRESSfit<sup>®</sup> for NOAH CD into the CD ROM drive.  
Note: The installation may automatically start when the CD is first inserted.
3. From the EXPRESSfit<sup>®</sup> for NOAH CD, double-click “setup.msi” / “setup.exe”.
4. Follow the on-screen instructions to install EXPRESSfit<sup>®</sup> for NOAH Fitting System software.

### Notes:

- If upgrading from a previous version of the EXPRESSfit<sup>®</sup> for NOAH Fitting System, it is important that you restart your computer after the installation.
- When installing the EXPRESSfit<sup>®</sup> for NOAH Fitting System on any version of the Windows Vista operating system, you must be present during the installation to acknowledge any prompts from the installation software. As a security measure, Windows Vista will automatically quit any installation that is not acknowledged after a period of time.

## EXPRESSfit<sup>®</sup> Standalone Fitting System

### Types of EXPRESSfit<sup>®</sup> Standalone Fitting System Installations:

#### Local Install

Patient fitting data is available only on the computer where the EXPRESSfit<sup>®</sup> Standalone software is installed. Other computers on the network cannot access the patient fitting data.

#### Network Server

Patient fitting data is available to the computer where the EXPRESSfit<sup>®</sup> Standalone software is installed, as well as other computers on the network. After installation, the Network Server computer must be configured to allow other computers on the network to access the EXPRESSfit<sup>®</sup> database. See the EXPRESSfit<sup>®</sup> section of our website for details.

#### Network Client

Patient fitting data is only available by accessing information stored on a Network Server. Data is not stored on the computer where the EXPRESSfit<sup>®</sup> Standalone software is installed.

### Steps to Install the EXPRESSfit<sup>®</sup>: Standalone Fitting System:

1. Insert the EXPRESSfit<sup>®</sup> Standalone CD into the CD ROM drive.  
Note: The installation may automatically start when the CD is first inserted.
2. From the EXPRESSfit<sup>®</sup> Standalone CD, double-click “Start\_Here.htm”.
3. Choose a type of EXPRESSfit<sup>®</sup> Standalone installation.  
See above for a description of the different types of EXPRESSfit<sup>®</sup> Standalone installations.  
Note: If Security Warning messages appear, click the “Run” button.
4. Follow the on-screen instructions to install the EXPRESSfit<sup>®</sup> Standalone Fitting System software.  
If installing a Network Server, see the instructions below to allow other computers on the network to access the EXPRESSfit<sup>®</sup> database.

### Notes:

- The Local Install and Network Server installation options may take up to 45 minutes.
- If prompted, it is important that you restart your computer during or after the installation.
- When installing the EXPRESSfit<sup>®</sup> Standalone Fitting System on any version of the Windows Vista operating system, you must be present during the installation to acknowledge any prompts from the installation software. As a security measure, Windows Vista will automatically quit any installation that is not acknowledged after a period of time.

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[www.sonici.com](http://www.sonici.com)

©2010, Sonic Innovations, Inc. All Rights Reserved.

The information and data contained in this guide is subject to change without notice.

This guide contains Sonic Innovations, Inc. proprietary information, and may not be copied or further distributed.

Windows 2000/XP/Vista are registered trademarks of the Microsoft Corporation.

NOAH and NOAHlink™ are registered trademarks of the HIMSA A/S in Denmark.