

Use with a Mobile Phone

Answer incoming phone calls	SoundGate Bluetooth® function must be active. When phone rings, press phone key on SoundGate to answer.	
When talking	Talk into SoundGate microphone (optimal distance is 12 inches/30 cm).	
Hang up the phone	Press SoundGate phone key again to end call.	
Make a call	Place call from your mobile phone like normal. Talk into the SoundGate microphone – do not press the SoundGate phone key.	
Adjust volume	Press the volume/program key up or down until the sound is comfortable.	
Turn hearing aid microphones ON/OFF	Press the volume/program key up or down for 2 seconds.	



Reconnection tip

Paired Bluetooth® devices find each other when turned on. If devices do not connect automatically, they must be connected manually. Always turn on your mobile phone before turning on the Bluetooth® function in the SoundGate.

Out of range handling of Bluetooth®


If the SoundGate is moved out of Bluetooth® range of the mobile phone, the connection will be lost. Moving the devices back into range should cause them to re-connect.

Use for remote control

Adjust volume	Adjust volume level up or down in the hearing instruments with a short press.	
Change program	Press and hold up or down for 2 seconds to go forward or backward in the program cycle. Note: This is only possible if the hearing instruments are configured for multiple programs. During audio streaming, it is not possible to shift to another program.	

SoundGate

Quick Guide

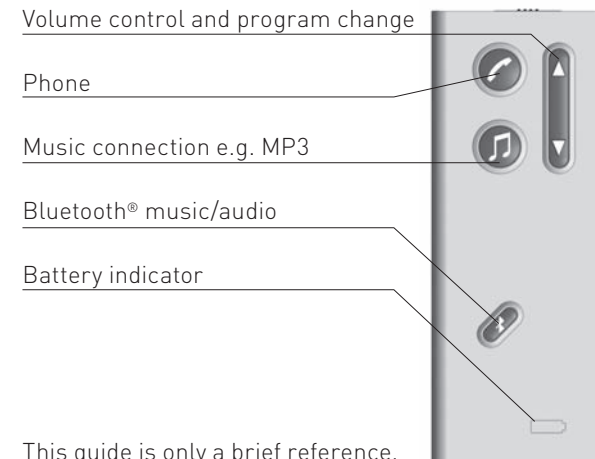

For more detailed information please refer to the SoundGate User Guide.

www.sonici.com

SONIC
innovations

Pairing to Bluetooth® Devices

Before using the SoundGate with a Bluetooth® mobile phone or audio source, the SoundGate must be paired to the device. Once a device is paired, it will be recognized by the SoundGate whenever the device is in range and the SoundGate Bluetooth® function is ON.

Prepare Bluetooth® device	Refer to the manual of the phone or audio device and prepare the device for Bluetooth® as instructed.
Set SoundGate into pairing mode	To activate pairing, press and hold the SoundGate Bluetooth® key for 7 seconds until the blue light flashes rapidly. Pairing mode is active for 120 seconds.
Activate pairing on Bluetooth® device	Activate the pairing process on the device. When successful, the word “SoundGate” will appear in the device display.
Enter pin code	The SoundGate code is 0000. Once entered, the device should be paired to the SoundGate. (The device may require that you manually activate the connection by selecting “Connect” in the display menu.)

Listen to Music/Audio

Audio sources can be connected to the SoundGate:

- > by wired input
- > by wireless Bluetooth® connection

Connecting to wired audio input

Use the audio cable and connect the larger end to the audio source (music player, etc.) and the smaller end to the plug in the bottom of the SoundGate.

Connecting to a Bluetooth® audio source

Pair desired device to SoundGate.

Listen to music via Bluetooth® Press the Bluetooth® key for 2 seconds. Then turn on the audio source.



Music/Audio Key

The table shows how to use the SoundGate for all types of audio streaming.

Listen to music or audio	Press the SoundGate music/audio key to play. Incoming phone calls can still be answered with a short press on the phone key.	
Stop the music/audio	Press the music/audio key again.	